

## BILE ACIDS CONTROLS SET

**REF** 450-22

For other languages  
Pour d'autres langues  
Für andere Sprachen  
Para otras lenguas  
Per le altre lingue

Para outras línguas  
Για τις άλλεςλώσεις  
For andre sprog



[www.trinitybiotech.com](http://www.trinitybiotech.com)

### ASSIGNED VALUES FOR BILE ACIDS CONTROLS

The assigned values are lot specific and can be found on the enclosed component list.

### INTENDED USE

Bile Acids Controls are designed for use in the assay of bile acids per **Trinity Biotech** Procedure No. 450 to verify procedure reliability. Set contains 3 x 5 ml each of a normal control and an abnormal control.

### REAGENTS

**BILE ACIDS CONTROLS SET, Catalogue No. 450-22**

**BILE ACIDS CONTROL, NORMAL, Catalogue No. B 5021, 3 x 5 ml**

**BILE ACIDS CONTROL, ABNORMAL, Catalogue No. B 6021, 3 x 5 ml**

Solutions contain bile acids in bovine serum base with sodium azide, 0.1%, as preservative.

### PRECAUTIONS:

Bile Acids Controls are for "in vitro diagnostic use". Normal precautions exercised in handling laboratory reagents should be followed. Dispose of waste observing all local, state and federal laws.

For professional use only.  
In case of damage, do not use.

Bile Acid Controls contains sodium azide which is HARMFUL. Harmful in contact with skin and if swallowed. Contact with acids liberates very toxic gas. Keep container tightly closed. Wear suitable protective clothing and gloves. Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. Avoid azide accumulation.

### PREPARATION:

Bile Acids Controls are supplied ready for use.

### STORAGE AND STABILITY:

Store Bile Acids Controls refrigerated (2-8°C). Vial labels bear expiration date.










### INSTRUCTIONS FOR USE

Bile Acids Controls should be treated in the same manner as test samples in the assay for bile acids.

### ORDERING INFORMATION

Catalogue No.	Item	Quantity
450-22	BILE ACIDS CONTROLS Contains 3 x 5 ml each of a normal and abnormal level	6 x 5 ml

### GUIDE TO SYMBOLS

 Consult instructions for use	 Temperature limit
 Catalogue number	 <i>In vitro</i> diagnostic medical device
 Use-by date	 Batch code
 <a href="http://www.trinitybiotech.com">www.trinitybiotech.com</a> eIFU indicator	 Acute toxicity
	 Manufacturer



Trinity Biotech plc,  
IDA Business Park  
Southern Cross Road  
Bray, Co. Wicklow  
Ireland  
Tel. 353 1 2769800  
Fax. 353 1 2769888  
[www.trinitybiotech.com](http://www.trinitybiotech.com)

USA ENQUIRIES:  
Trinity Biotech USA  
2823 Girts Road,  
Jamestown, NY 14701.  
Tel. 1800 325 3424  
Fax. 716-486-1990

