

# Bartels Viral Respiratory Screening and Identification Kit Non-Immune Mouse Antibody

**REF** B1029-86C

1 x 10 mL

Pour d'autres langues  
Für andere Sprachen  
Para otras lenguas  
Per le altre lingue  
Dla innych języków

Para outras línguas  
Για τις άλλεςλώσσες  
Für andra språk  
For andre språk



[www.trinitybiotech.com](http://www.trinitybiotech.com)

## INTENDED USE

Intended as a replacement component, Bartels Non-Immune reagent is used as a negative control during the VRK procedure. Bartels Viral Respiratory Screening and Identification Kit (Catalogue Number B1029-86) is intended for use in cell culture confirmation and direct specimen detection. The screening reagents and the individual antibodies directed against adenovirus, influenza A, influenza B, parainfluenza type 1, 2, and 3, and respiratory syncytial virus (RSV) can be used for cell culture confirmation. The influenza A, influenza B, parainfluenza type 1, parainfluenza type 3, and respiratory syncytial virus (RSV) individual antibodies can be used for direct specimen detection. Specimens found to be negative on direct specimen examination should be confirmed by cell culture confirmation. Monoclonal antibodies directed against influenza and parainfluenza viruses are type specific, while monoclonal antibodies directed against adenovirus and RSV will detect all identified serotypes of the respective virus. The Bartels Viral Respiratory Screening and Identification Kit uses indirect fluorescent antibody technology.

**Note:** To be used as a replacement component for Bartels Viral Respiratory Screening and Identification Kit, catalogue number B1029-86. This package insert provides summary information only. For more detailed information refer to package insert part number B1029-86-29, (catalogue number B1029-86), or contact Technical services in the U.S. at 1-800-325-3424, or outside the U.S. at (353) 1 276 9800.

## CONTENTS

**For in vitro diagnostic use only.**

**Non-Immune Mouse Antibodies, 1 x 10 mL, B1029-86C**

Affinity purified non-immune mouse antibodies are diluted in phosphate buffered saline (0.15 M sodium chloride and 0.01 M sodium phosphate) with 0.75% protein stabilizer. Sodium azide and ProClin® 950 are added at concentrations of 0.1% as preservatives. Store at 2-8°C.

## WARNINGS AND PRECAUTIONS

- For in vitro diagnostic use only.**
- Any reagent containing sodium azide or ProClin® 950 should be evaluated for proper disposal. Sodium azide may react with lead and copper plumbing to form highly explosive metal azides. If products containing sodium azide are discarded into a drain, flush with a large volume of water to prevent azide build-up. Check with regulatory agencies to determine at what concentration sodium azide may cause a product to be regulated as a hazardous waste. MSDS for sodium azide and ProClin® 950 are available by phoning a Technical Service Representative in the U.S. at 1-800-325-3424, or outside the U.S. at (353) 1 276 9800.
- Reagent is supplied at working strength. Dilution of the reagent will decrease sensitivity.
- Reagent should **not** be used beyond its expiration date.
- Microbial contamination of reagent may cause a decrease in sensitivity.
- All specimens and materials should be considered potentially infectious and handled in a manner which prevents infection of laboratory personnel. Decontamination is most effectively accomplished with a 0.5% solution of sodium hypochlorite (1:10 dilution of household bleach).
- Never** pipette reagents or clinical samples by mouth.
- Avoid splashing, generation of aerosols, and contact with broken skin when processing clinical specimens or virus stocks.
- Do **not** substitute reagents from other manufacturers.
- Incubation times or temperatures other than specified in the test instructions may give erroneous results.
- Cross-contamination of reagents could cause false results. If staining multiple patients on the same slide, keep each specimen within the perimeter of its well to insure that reagent will not be drawn out of the well when stained.
- Do **not** allow wells to dry once assay has begun.
- Reusable glassware must be washed and thoroughly rinsed free of all detergents.
- Due to the affinity of adenovirus for the ocular area, hand to eye contact should be avoided during processing of respiratory specimens and when performing diagnostic tests.
- Use aseptic technique and sterile equipment and materials for all tissue culture procedures.




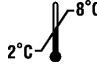


## STORAGE AND STABILITY

Reagent component will retain its full potency until the expiration date shown on the label of each vial when stored at recommended temperatures.

## TECHNICAL INFORMATION

For further information or assistance, contact: Technical Services in the U.S at 1-800-325-3424 or outside the U.S. at (353) 1 276 9800.

## GUIDE TO SYMBOLS

 Consult Instructions for Use	 Use by			
<b>REF</b> Catalogue number	<b>IVD</b> For in vitro Diagnostic Use			
<b>CONT</b> Contents	 Manufacturer			
<table border="1"> <tr> <td>REAG</td> <td>VRK</td> <td>NIMA</td> </tr> </table> Non-immune mouse antibodies	REAG	VRK	NIMA	<b>LOT</b> Batch code
REAG	VRK	NIMA		
 Store at 2-8°C				
	Trinity Biotech Plc Bray, Co. Wicklow, Ireland Tel. 353 1 2769800 Fax. 353 1 2769888 www.trinitybiotech.com			
	<b>USA ENQUIRIES:</b> Trinity Biotech USA 2823 Girls Road, Jamestown, NY 14701. Tel. 1800 325 3424 Fax. 716 488 1990			
B1029-86C-29 Rev C 09/2015				